





# Transmittal Page


Customer Stamp

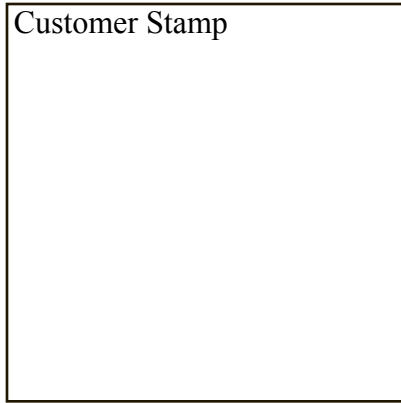


Architect Stamp

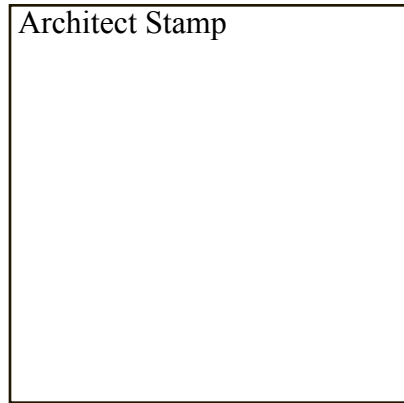

Mechanical Stamp

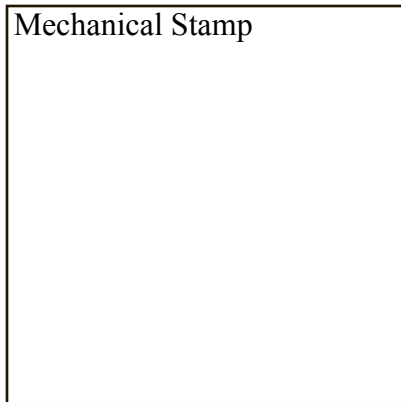

Electrical Stamp

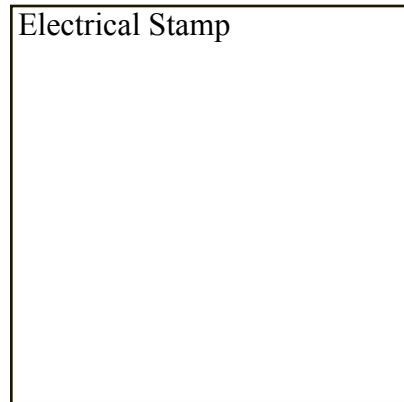

## **IMPORTANT**

Amico cannot commence manufacturing until one completely “Approved” copy of these Shop Drawings are returned to our attention.

THIS SHOP DRAWING BOOKLET INCLUDES:

	GENERAL PRODUCT SPECIFICATION(S)
	MANUFACTURING VERIFICATIONS
	ELEVATION DRAWING(S)
	MANUFACTURER WARRANTY STATEMENT
	INSTALLATION INSTRUCTIONS
	LISTING OF APPROVALS

MANUFACTURING OF THE PRODUCT(S) COVERED BY THIS SHOP DRAWING BOOKLET WILL NOT COMMENCE UNTIL ALL VERIFICATIONS ARE INITIALLED WHERE REQUIRED, AND INFORMATION PROVIDED WHERE REQUIRED.

**\*\*\*IMPORTANT\*\*\***

The following statement forms an integral part of this shop drawing submittal. By approving and returning this submittal, the customer acknowledges and accepts the following statement.

**THE INFORMATION HEREIN FORMS A “MANUFACTURING BLUEPRINT”, AND SHALL TAKE PRECEDENCE OVER ALL OTHER DOCUMENTATION, INCLUDING BUT NOT LIMITED TO, PROJECT SPECIFICATIONS, ADDENDA, AND DRAWINGS.**

## Specifications – LDRP wall

Amico Corporation shall manufacture the Amico Alert-2 LDRP, in accordance with these shop drawings and documents. The complete LDRP wall shall be UL listed.

### 1. Basic Construction

**Enclosure** shall be constructed of decorative laminate with treated wood. The unit shall be factory assembled to include mechanical and electrical components as shown on the plans and specifications and wired in accordance with Amico Corporation's shop drawings as herein indicated. Access panels shall be removable for access to individual components mounted within the console sections of the wall unit. The electrical contractor provides primary electrical conduit runs to the unit. All terminations shall be in the wall compartments. All secondary wiring within the wall unit enclosure shall be enclosed in fixed wiring raceways and conduits. All wiring shall be marked, color-coded and tie wrapped to facilitate easy circuit identification.

Removable panels shall be finished with high-pressure plastic laminate on the outside face. The panel allows users to access services, hook ups, and easier upgrades. The hospital or architect can select from Amico standard color selections for the plastic laminate.

### 2. Components

**Medical gas station outlets.** Medical gas outlets shall be as indicated in plans and specifications. Amico Corporation will factory install all outlets, manifolded and tested in accordance with the outlet manufacturer's requirements. Each outlet, piping and manifold shall be pressure tested for 24 hours. The mechanical contractor will bring the primary supply lines of the medical gases to the singular terminal connections at locations shown on the drawings.

**Power outlets.** Receptacles shall be the type and quantity as shown on the drawings and specifications. Unless otherwise noted 2 poles, 3

wires, rated at 20 amperes, 120 Volt, Hospital Grade shall be provided.

**IV Hooks.** Are stored out of the way for ease of accessibility.

**Sphygmomanometer.** Shall be included with the head wall if specified and installed by the contractor. The unit shall be supplied with basket and mounting hardware.

**Vacuum Bottle Slide.** Shall be a 4 inch anodized aluminum device with bottom stop. It shall be installed on the wall as specified on drawings.

**Internal ground bus.** A ground bus shall be included with a minimum of #6-screw type. The ground bus assembly shall also provide a positive means for grounding the wall.

**Switches.** Shall be as shown on drawings, 120 VAC 20 Amp "Specification" Grade.

**Grounding jacks.** Externally accessible grounding jacks shall be provided in the LDRP wall unit, as shown in the shop drawings. The solid brass receptacle shall be enclosed in a non-conductive housing, shall be spring-loaded and shall be manufactured to the requirements of NFPA 99.

**Lighting fixture.** The indirect light provides a soft warmth feeling in the room. The direct light will provide an abundant source of lighting. Amico Corporation does not provide lamps. Light can be switched thru Amico Low Voltage Controller if required.

### 3. Provisions

**Nurse call equipment.** Shall be furnished and installed by others as indicated in the plans and specifications. Amico Corporation shall provide a labeled pull cord for use by the nurse call installer.

**Code Blue.** Equipment shall be furnished and installed by others as indicated in the plans and

specifications. Amico Corporation shall provide a labeled pull cord for use by contractor.

**Telephone Provisions.** Shall include a 1/2" conduit with "Pull-cord" to junction box. Telephone Jack shall be installed and wired by contractor or local telephone company.

#### **4. Accessories**

**Accessories** shall be provided in the quantities and types as shown in the submittal. Others shall install accessories in the locations as determined by the end user.

#### **5. Installation**

**Mounting** of the product shall be the responsibility of the installing contractor. This includes receiving, storage, erection, bracing, clean up, touch up, carton disposal etc. This contractor shall supply all necessary installation material including such items as tools, fasteners, caulking and electric lamps not supplied by Amico Corporation.

**The electrical contractor** shall be responsible for all electrical hook-up at service connection locations plus interconnect wiring on multi-section units. After the installation is complete the electrical contractor shall test equipment function plus electrical receptacles and grounding in accordance with local electrical code requirements.

**The medical gas contractor** shall be responsible for piping and hook-up of all medical gas services. In addition, hook-up of piping between sections on multi-section units shall be the responsibility of this contractor. The medical gas contractor shall be responsible for purging, pressure testing, gas identification and system certification in accordance with NFPA 99 or CSA-Z305.1.

**Accessory items** shall be installed in accordance with the manufacturer's instructions and shall be the responsibility of this section. The installation shall take place under the direction of the hospital.

*Note: All Specifications are nominal and subject to change without notice.*

# AMICO VERIFICATIONS

WE CANNOT BEGIN PRODUCTION WITHOUT THIS INFORMATION

Please call \_\_\_\_\_ with any questions.  
Tel: \_\_\_\_\_  
Fax: \_\_\_\_\_

## 1) GENERAL

- Please initial thus verifying the quantity shown on each elevation drawing is correct. Additionally, where applicable, please identify in **RED INK** the number of “opposite” units required for each elevation (“opposite” shall be defined as mirror image”).
- Please confirm the quantity of “As Shown” and “Mirror Image” units required for each elevation:

<u>Total Qty</u>	<u>“As Shown”</u>	<u>“Mirror Image”</u>
------------------	-------------------	-----------------------

- Please initial thus verifying that all service and provision components are located as required on each elevation drawing. Please mark any corrections on the elevation drawing(s) in **RED INK**.
- All product covered by these verifications and related elevation drawings shall be built to manufacturer’s standard specification (copy enclosed); exceptions noted herein.
- Please provide the required on-site delivery date here: \_\_\_\_\_

Standard delivery shall be 8 - 10 weeks from factory receipt of “APPROVED” shop drawings (containing additional information where requested) and any applicable Division 15 & 16 supplied components.

Delivery date identified above shall not extend beyond the quoted delivery by date of : \_\_\_\_\_

- When elevation drawings reference specific room numbers, the manufacturer will endeavor to label product-shipping cartons with the appropriate room number. It remains, however, the sole responsibility of the installing contractor to ensure all products are installed in the correct location(s) within the correct room(s).

## 2) MOUNTING HARDWARE / SPECIAL REQUIREMENTS

- Headwalls are shipped complete with a wall-mounting bracket, from which the headwall is suspended prior to final wall attachment. Hardware required to secure mounting bracket to wall, and headwall to wall, shall be supplied by others to suit wall type.
- Anti-sway ceiling structure required for installation of Power Columns shall be designed, supplied and installed by others.

## 3) PRODUCT COLORS



*If circuit schedule is not provided, circuits will be factory tagged and identified.*

- Line voltage devices shall be factory supplied, installed, and wired with #10 or #12, stranded copper wire (THHN, 600 volt insulation) for hot and neutral (black and white). Grounds shall be #10, type R, THHN stranded copper wire (green). All ground conductors shall be installed in conduit.
- Please initial thus verifying the power source is 120/208 volts.
- Please select one of the following:
  - Circuits shall be wired with shared neutrals.
  - Circuits shall be wired with individual neutrals.
- Please select one of the following:
  - All receptacles shall be mounted with ground pin down.
  - All receptacles shall be mounted with ground pin up.
- Provisions for “future” line voltage electrical devices shall consist of back boxes, conduit from back boxes to junction box, and a blank stainless steel cover plate.
- All line voltage device wiring shall be terminated within a junction box located within the headwall as indicated on the elevation drawings. All connections to hospital services shall be made by others.
- Overbed light referenced. Hospital grade overbed lights should include a safety switch leg for integration within the power circuit for an electric bed receptacle. When provided, wiring of such circuit shall be the responsibility of others. If provided, please identify in **RED INK** which receptacle on the headwall will be connected to this circuit.

*When wiring a bed receptacle to an overbed light safety switch leg, we recommend using a simplex receptacle and labeling it “for bed use only, not for life support equipment”.*

6) LOW VOLTAGE PROVISIONS

- Provisions only, consisting of back boxes and conduit, will be supplied for all low voltage devices. Devices shall be supplied and site installed by others. Please circle the required opening and back box gang size required for each of the following low voltage provisions:

<u>Device</u>	<u>Opening Size (gang)</u>				<u>Back Box Size (gang)</u>			
Nurse Call (NC)	1	2	3	4	1	2	3	4
Nurse Assist (NA)	1	2	3	4	1	2	3	4
Code Blue (CB)	1	2	3	4	1	2	3	4
Telephone/Data (T)	1	2	3	4	1	2	3	4

**SHOP DRAWINGS RECEIVED WITHOUT THIS INFORMATION WILL BE RETURNED. PRODUCTION WILL NOT COMMENCE WITHOUT THIS CRITICAL INFORMATION.**

- Provisions for “future” line voltage electrical devices shall consist of back boxes, conduit from back boxes to junction box, and a blank anodized aluminum cover plate.

7) MEDICAL GASES/VACUUM

— Medical gas terminal units (outlets) shall be \_\_\_\_\_ console style outlets

— Headwall manufacturer shall provide all medical gas terminations with either 1/2" or 3/4" diameter type "K" seamless copper pipes, factory installed. Factory installed copper pipes shall be cut to length and terminate as indicated on elevation drawings.

— Connection between hospital pipeline systems and headwall pipe stubs shall be brazed on site by others, in accordance with local building codes, project specifications, and NFPA 99 and /or CAN CSA Z305.1 (latest edition).


8) OTHER

August 28, 2000

## **WARRANTY POLICY**

1. Amico Corporation warrants its Medical Gas Equipment to be free from defects in **material** and **workmanship** for a period of twelve (12) months from the date of shipment. Within this period Amico will repair or replace any part on site, or at the factory, which is proven to be defective at Amico's cost.
2. Furthermore, Amico will warrant its **material** to be free from defect for an additional period of four (4) years (five (5) years from the date of shipment). Within this period, Amico will replace any **part**, at no charge, which is proven to be defective. Shipping and Installation costs after the first twelve (12) months will be borne by the Customer.
3. This warranty is valid only when the product has been properly installed according to Amico specifications, used in a normal manner and serviced according to factory recommendations. It does not cover failures due to damage which occurs in shipments or failures which resulted from accidents, misuse, abuse, neglect, mishandling, alteration, misapplication or damage that may be attributable to acts of God.
4. Amico shall not be liable for incidental or consequential damages resulting from the use of the equipment.
5. All claims for warranty must first be approved by Amico's Service Department ([service@amico.com](mailto:service@amico.com) or 1-877-462-6426). A valid Return Goods Authorization (RGA) number must be obtained from Amico prior to commencement of any service work. Warranty work, which has not been pre-authorized by Amico, will **not** be reimbursed.


# LDRP MOUNTING CONCEPT

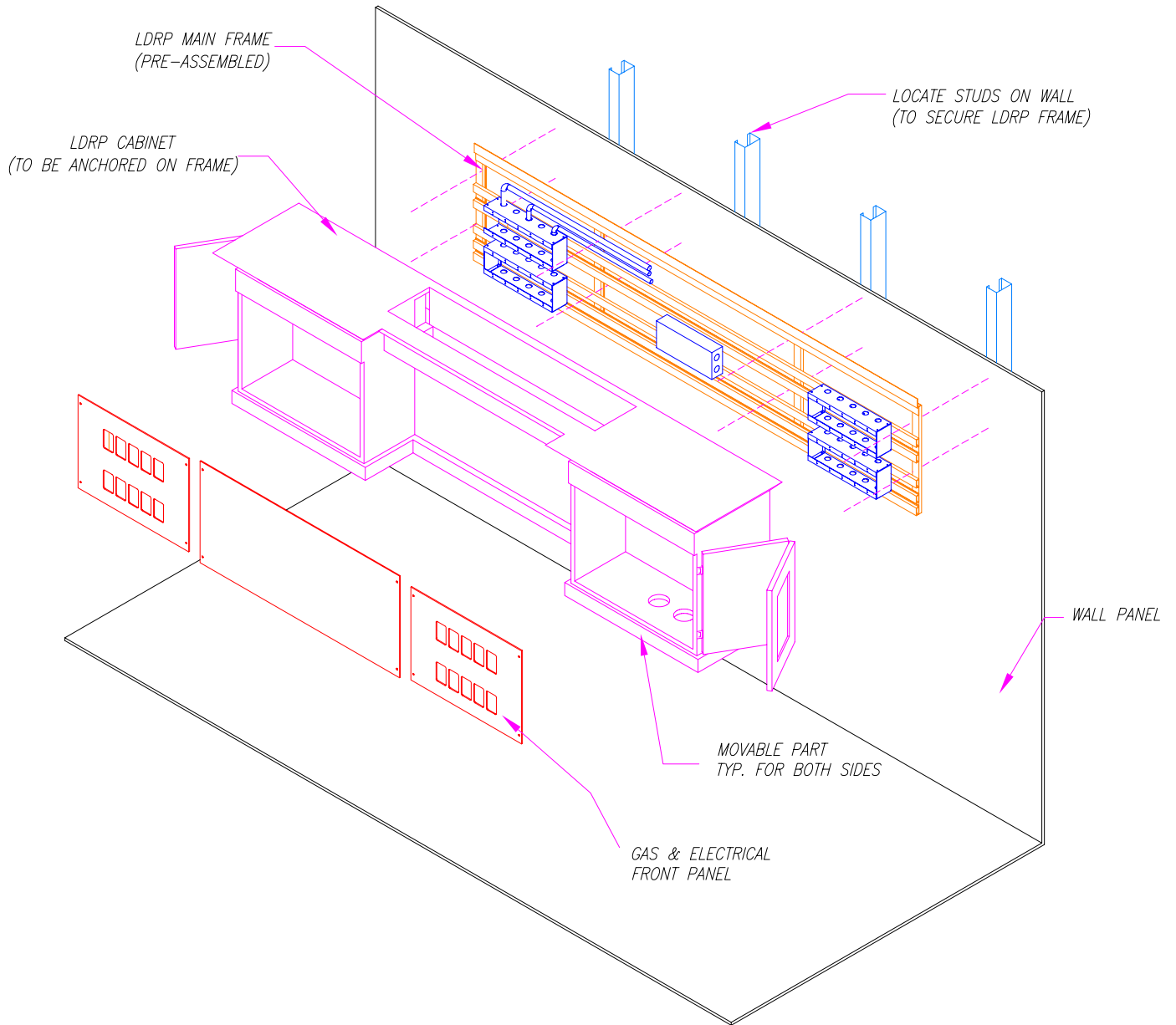

## LDR Installations instructions

**IMPORTANT: DO NOT SIT THE CABINET ON THE MOVABLE BOTTOM TRAY. ALWAYS LAY CABINET ON ITS BACK!!!!!!**


1. Secure the frame to the wall.
2. Remove electrical access panel(middle).
3. Hang up the cabinet over the frame. Center the cabinet on the frame. Cabinet openings from left and right sides should match with boxes from the frame.
4. Do all necessary connections for the light and bottom micro switches (see attached diagram).
5. Have left and right front panels for your unit handy. Pull all electrical devices out of the boxes. Put the front panel on the cabinet and match the cut-outs with the devices positions. Do this step for both sides of the cabinet. Adjust cabinet position for a better alignment if necessary.
6. Holding the front panel on the correct position, surface mount the devices in place and then secure the cover to the cabinet. (Drill pilot holes and attach with screws provided)
7. Install devices cover plates.
8. Anchor the cabinet to the frame using middle brackets attached to the cabinet.
9. Install the middle panel in place.
10. Insert the trim caps where necessary.
11. Install blood pressure device.
12. Install vacuum slides to the bottom using screws provided.

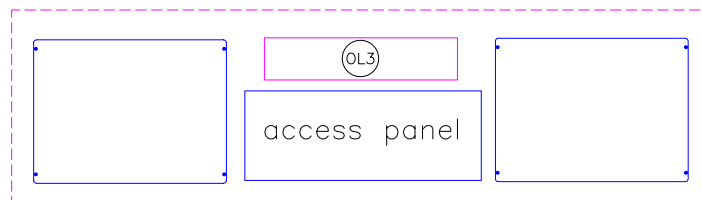

500 North Dearborn Street, Suite 600  
Chicago, Illinois 60610  
Phone: (773) 399-2000  
Fax: (773) 399-2000  
Web: www.ul.com  
UL is a registered service mark of Underwriters Laboratories Inc.



AMICO CORP  
MR S IAPS  
21-121 GRANTON DR  
RICHMOND HILL  
ON L4B 3N4 CANADA

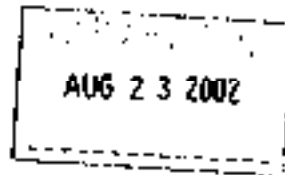

RE: Project Number(s) - 02NK39938

Your most recent Certification is shown below. You may also view this information, or a portion of this information (depending on the product category), on UL's Online Certifications Directory at [www.ul.com/database](http://www.ul.com/database). Please review the text and contact the Conformity Assessment Services staff member who handled your project if revisions are required. For instructions on placing an order for this information in a 3 x 5-inch format, you may refer to the enclosed order form for UL Card Service.

KEZR August 12, 2002  
Prefabricated Medical Headwalls and Medical Supply Units

AMICO CORP  
21-121 GRANTON DR RICHMOND HILL, ON L4B 3N4 CANADA

E192931

Medical headwall - Medical headwall with gas stations

Medical headwall, Model W-BED-LOCATOR Bed locator.  
Medical headwalls with gas stations, Models C-COL-MAN, C-COL-PNE, C-COL-STA, C-COL-STA-LI Ceiling solutions.  
Model E-CON-XX Console unit.  
Model F-PEDESTAL Floor pedestal.  
Model W-BHW-XX-XX-XX Horizontal wall.  
Models W-PC-XX-L-XXX, W-PC-XX-XX-XXX Power outlets.  
Reduction Collection units, Labour Delivery Rooms (LDR).  
Model W-FW-XXFL-XXX Wettable wall.  
XX - Represents the diameter in inches.  
XXX - General Care, Special Care or ICU.

LOOK FOR LISTING MARK ON PRODUCT