



## NCG Chemetron Outlet Retro-Fit Manual

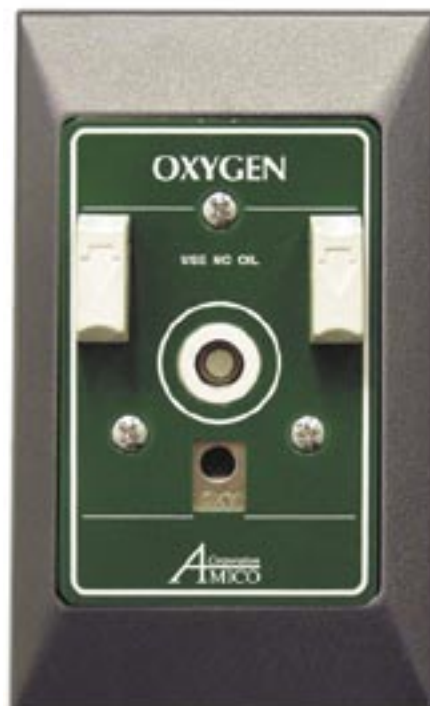


**Step 1:**

Remove all secondary check parts from the existing outlets. Replace the NCG rubber seat with the Amico #15 O-Ring.

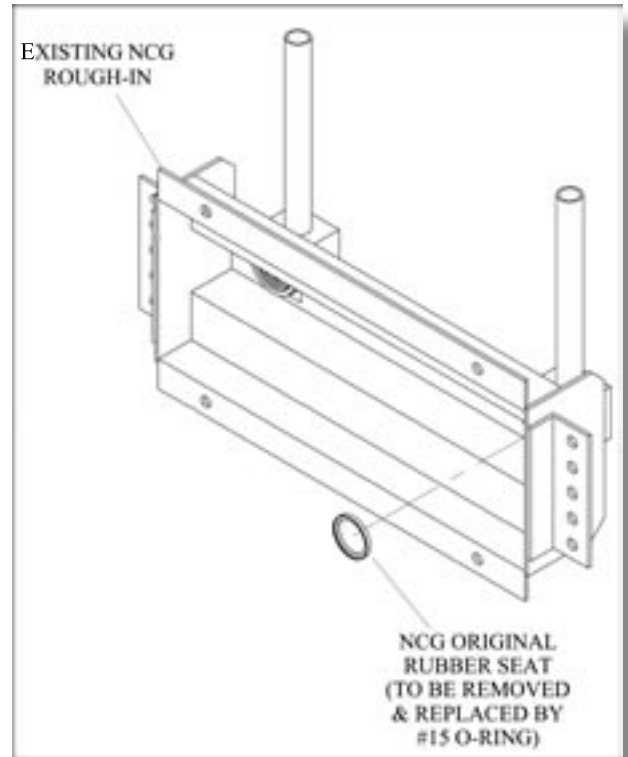

**Step 2:**


Thread the Amico Retro-Fit rough-in assembly to the existing NCG rough-in.

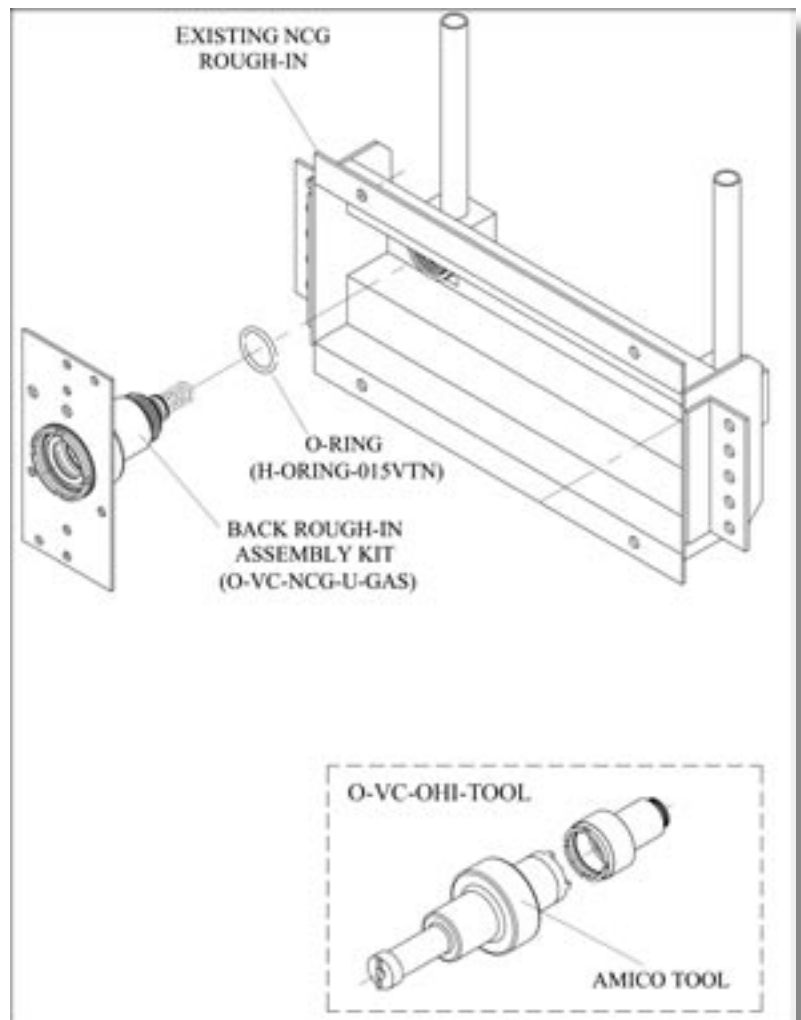


**Step 3:**

Use the Rivet provided to attach the Upgrade bracket to the existing console to ensure it is gas specific.

NOTE: Ensure the Amico rough-in and the NCG rough-in are the same gas.

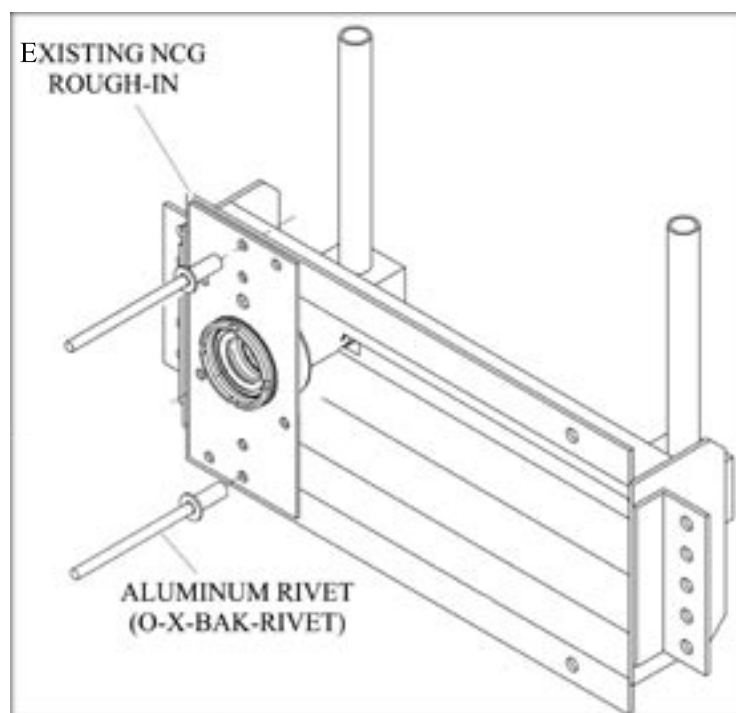**Step 4:**

Place the pre-assembled filler frame & face plate over the wall outlet and insert the latch valve assembly through the face plate and fasten with the latch valve screws.

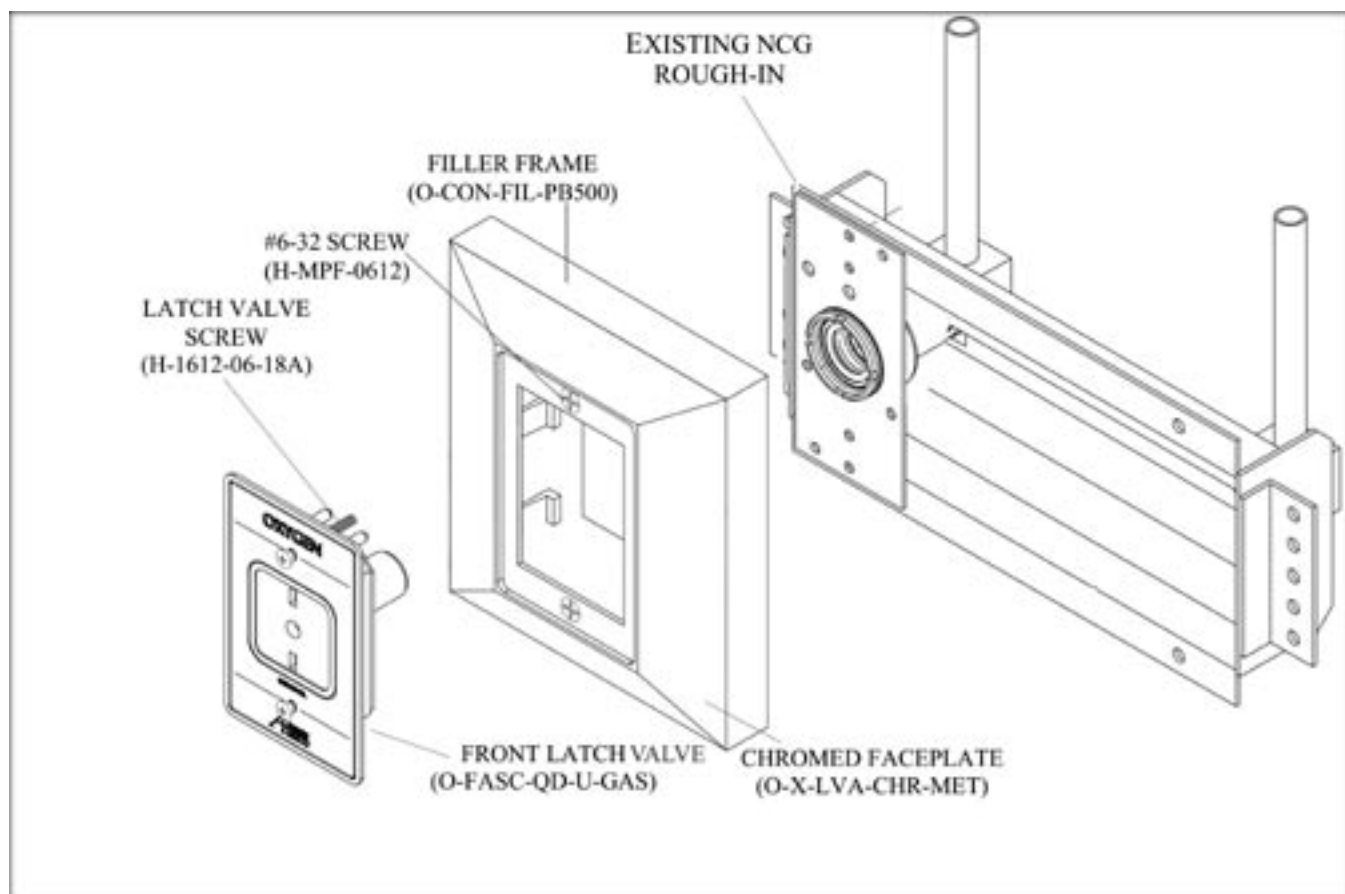



14 Madison Rd., Fairfield, NJ 07004 USA  
85 Fulton Way, Richmond Hill, ON L4B 2N4 Canada

Tel: 1-877-GO AMICO (1-877-462-6426)  
Fax: (905) 764-0862 [www.amico.com](http://www.amico.com)