




MedæS Integrated Alarm Retro-Fit Manual

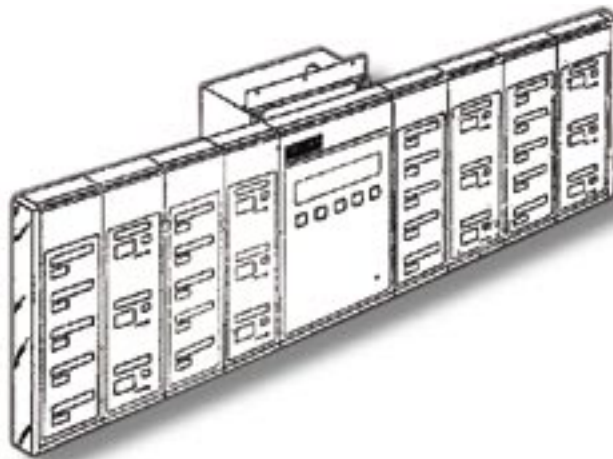

Product Specifications

The Amico retro-fit alarm will allow an existing Medæ's Integrated alarm to be replaced without removing the existing back box and pipe-stubs.

The filler frame enables the Amico Alert-2 Microprocessor alarm front assembly to sit onto the wall of the existing alarm panel. The filler frame acts like a spacer between the Integrated box and the front assembly. Please make sure that the gas is turned off before the retro-fit alarm is installed.

Installation Instructions


Remove all internal components of the Medæ's Integrated alarm (mother board, power supply, sensors and all wiring). Remove the front frame and the display modules from the back box.



Ensure that the back box is clean of ALL screws, studs, stand-offs and insert tabs. Bend the lower front lip of the alarm box. Ensure that the lips is flat.




Add the power supply to the far left corner of the back box assembly. Use 4 tapping screws to hold the power supply into the box. Use tapping screw for grounding wire.

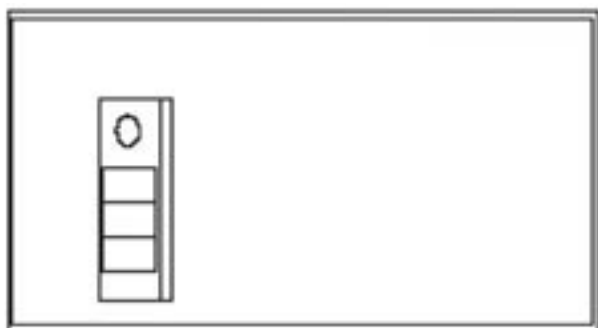


Remove the DISS fitting from the sensors that are mounted remotely.

Connect into place the Amico Alert-2 sensor, which contains a DISS fitting (nut and nipple).

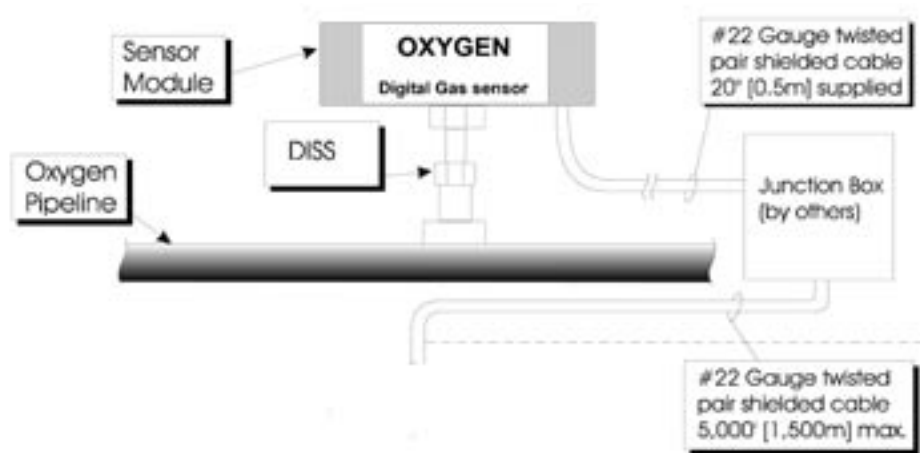

Add the Amico filler frame to the wall surface. Ensure that the box on the wall is in the center of the filler frame.

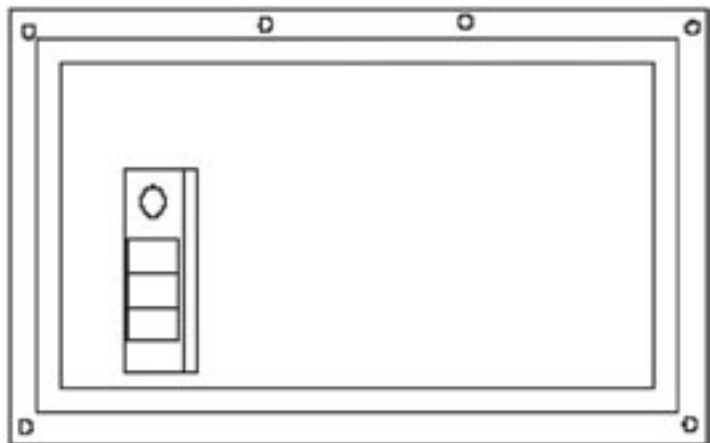

Install the front assembly and follow the "Installation & Operation Manual of the Alert-2 Alarm". Also supplied with the Retro-Fit Alarm.





14 Madison Rd., Fairfield, NJ, 07004, USA
85 Fulton Way, Richmond Hill, ON, L4B 2N4, Canada

Phone: 1-877-GO AMICO (1-877-462-6426)
Fax: (905) 764-0862 www.amico.com