





Ohio Gauge Alarm Retro-Fit Manual

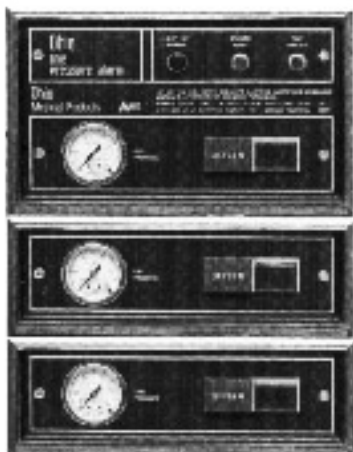


The Amico retro-fit alarm will allow an existing Ohio alarm to be replaced, without removing the existing back box and pipe-stubs.

The filler frame enables the Amico Alert-2 Microprocessor alarm front assembly to sit on the wall of the existing alarm panel back box. The filler frame acts like a spacer between the Ohio box and the front assembly. Please make sure the power source at the breaker panel is off before replacing the power supply. The gas supply may also be required to be shut off.

Remove all internal components of the Ohio alarm (mother board, power supply, sensors and all wiring). Remove the front frame and the display modules from the back box.

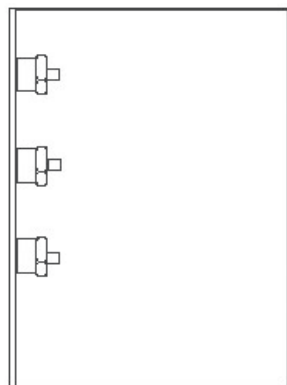


Ensure that the back box is clear of ALL screws, studs, stand-offs and insert tabs.

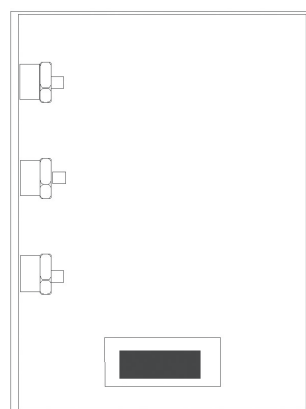


Attach the power supply to the bottom of the back box assembly. Use 7 tapping screws to hold the power supply assembly in place in the box. Tapping screws should also be used for grounding wires. Please note, tapping screws are not supplied by Amico.

Connect into place the ALERT-2 sensor, which contains the male quick-connect fitting, into the, already installed, hansen female fitting. NOTE: You will have to use flexible tubing to off-set the sensors.

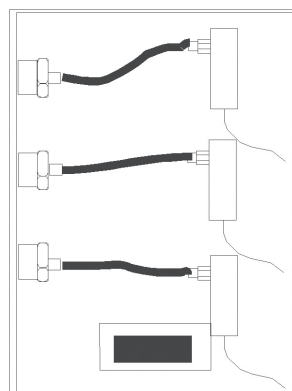

Attach the Amico filler frame to the wall surface. Ensure that the box on the wall is centered to the top of the filler frame (this may vary based on spacing). Attach the bottom plate to the bottom of the alarm box to cover the gap.

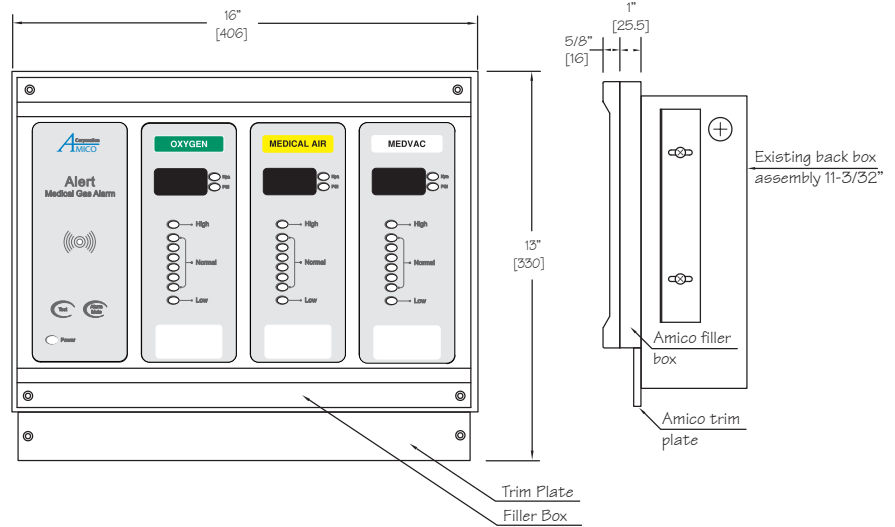

Install the front assembly and follow the "ALERT-2 ALARM INSTALLATION & OPERATION MANUAL"

(NOTE: The Ohio Alarm is supplied with both the Alert-2 and Retro-fit Alarm manual.)

A complete Alarm assembly conversion from Ohio to Amico.





14 Madison Rd., Fairfield, NJ 07004 USA
85 Fulton Way, Richmond Hill, ON L4B 2N4 Canada

Tel: 1-877-GO AMICO (1-877-462-6426)
Fax: (905) 764-0862 www.amico.com