



**CHEMETRON GAS
CONTROL PANEL
RETRO-FIT MANUAL**

Before



After

Chemetron Gas Control Panel.
Remove screws from panel.



Remove all internal components.



Mount the filler frame to the wall using the appropriate hardware for the type of wall you are mounting to. There are 6 holes provided for mounting the frame, 3 on top and 3 on the bottom.

Note: The frame can not be mounted on the wall centered on the existing back box. The retrofit is shipped with the filler frame attached to the control panel. Before you take these apart, push the assembly into the existing back box and position it into place and mark off the corners of the frame on the wall. These lines will be your reference for mounting the frame to the wall.



The pigtail adapter and the reducer bushing provided separately with the retrofit kit should be secured to the input and output in the back box as shown beside.



The pigtail that is supplied separately in the retrofit kit is for the output. Secure the male end to the reducer bushing on the output side in the back box.



Secure the male end of the output pigtail to the output on the back of the control panel. Secure the input pigtail, coming from the back of the control panel, to the input in the back box. Push the whole assembly gently into the back box and secure the panel to the filler frame using the #8 stainless steel machine screws provided with the kit. Please note the position of the pigtails. Make sure you position them this way or you may have trouble fitting the panel onto the frame.





14 Madison Rd., Fairfield, NJ 07004 USA
85 Fulton Way, Richmond Hill, ON L4B 2N4 Canada

Tel: 1-877-GOAMICO (1-877-462-6426)
Fax: (905) 764-0862 www.amico.com