

DIAMOND 1

RETRO-FIT MANUAL

Before



After


Ohmeda Diamond 1 Outlets.
Remove screws from Facia.



Remove all internal components so that the threads in the back are exposed.
Note the threads on the Diamond 1 conversion match Diamond 1 rough-in body.



Screw Diamond 1 conversion into rough-in body with the conversion tool.

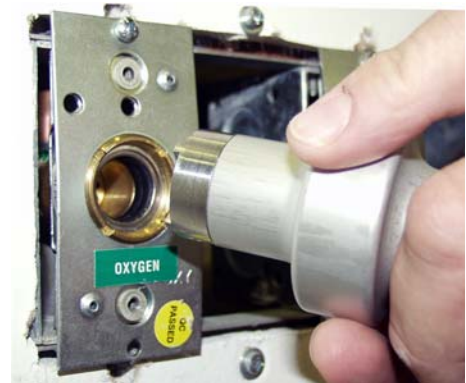

Rivets are popped in to ensure gas specific indexing.



A filler frame is placed over the conversion to hide any gaps and allow correct spacing from the outlet backs to the outlet fronts (this is like a gasket).



A 3.25" wide chrome frame is placed over the filler frame.



The Amico latch valve is then placed over the frame and sandwiches the whole assembly together with the latch valve screws.



A complete Diamond 1 3 Gang assembly conversion from Ohmeda to Amico.



A *Corporation*
AMICO 14 Madison Rd., Fairfield, NJ 07004 USA
85 Fulton Way, Richmond Hill, ON L4B 2N4 Canada

Tel: 1-877-GOAMICO (1-877-462-6426)
Fax: (905) 764-0862 www.amico.com