




DIAMOND 2

RETRO-FIT MANUAL

Before

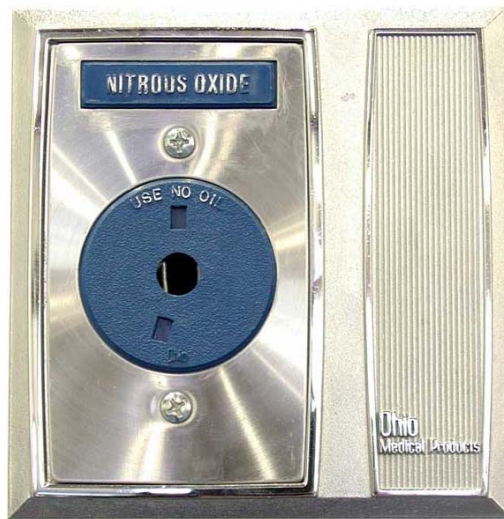

After

Diamond 2 Retro-Fit

Note: This application is for a complete conversion.

Ohmeda Diamond 2 & 3 Outlet.
Remove screws to expose rough-in.



Remove all components from existing rough-in.



Insert sleeve with o-ring and other components as seen below.



Amico Diamond 2 Conversion Kit.



Retaining Ring



Washer



Seat



Check Cap



Spring




Sleeve with O Ring

Place the chrome 4.5" frame over the rough-in and screw in the latch valve.



A completed Diamond 2 assembly conversion from Ohmeda to Amico.

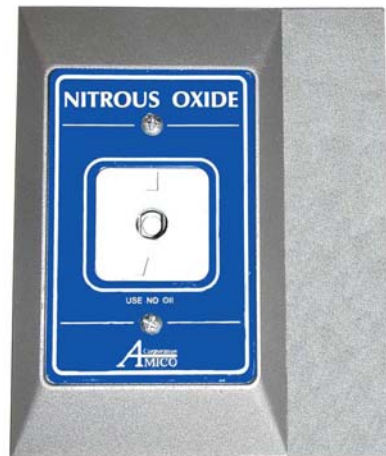

A *Corporation*
AMICO 14 Madison Rd., Fairfield, NJ 07004 USA
85 Fulton Way, Richmond Hill, ON L4B 2N4 Canada

Tel: 1-877-GOAMICO (1-877-462-6426)
Fax: (905) 764-0862 www.amico.com