

Diamond 3 Retro-Fit Manual



This Retro-Fit easily installs and should only take about 10 minutes. There is no need to shut off the gas. Please note that this application only Retro-Fits the front assembly and no parts will be sent along for the back rough-in. If you require a Retro-Fit to change the back rough-in you'll need to order the Diamond 2 Retro-Fit.

Installation Instructions

Ohmeda Diamond 2 & 3 Outlet.
Remove screws from front facia.



Front View

Amico has made this latch valve compatible to the existing rough-in. The barrel is shorter and the diameter is wider. Note the indexing of the pin is on the bottom.



Side View

Screw in the Amico Diamond 3 Latch Valve and 4.5" chrome frame.



A completed Diamond 3 assembly conversion from Ohmeda to Amico.





14 Madison Rd., Fairfield, NJ 07004 USA
85 Fulton Way, Richmond Hill, ON L4B 2N4 Canada

Tel: 1-877-GO AMICO (1-877-462-6426)
Fax: (905) 764-0862 www.amico.com