



Installation Instructions

bed locator




note

Please see Figure A when referring to installation instructions.

- 1 Remove all stainless steel face/cover plates.
- 2 Remove plastic bed locator shell (8 screws in total).
- 3 Using shop drawings for rough-in locations, ensure rough in services to connect to bed locator rough-in boxes are correct.
- 4 Mount the bed locator backing plate to wall structure. Use the three upper mounting holes (see [Figure A](#)) to attach, using hardware (by others) compliant to applicable building codes. Mounting holes are recommended to be at 25" above finished floor.
- 5 Connect all services to bed locator at rough-in locations. When done, attach backing plate to wall using lower mounting holes as in step 4.
- 6 Hang the plastic bed locator shell over the metal backing plate using its lip on the top.
- 7 Secure bed locator shell to backing plate by inserting screws removed in [step 2](#).
- 8 Re-attach all stainless steel face/cover plates removed in [step 1](#).

Figure A

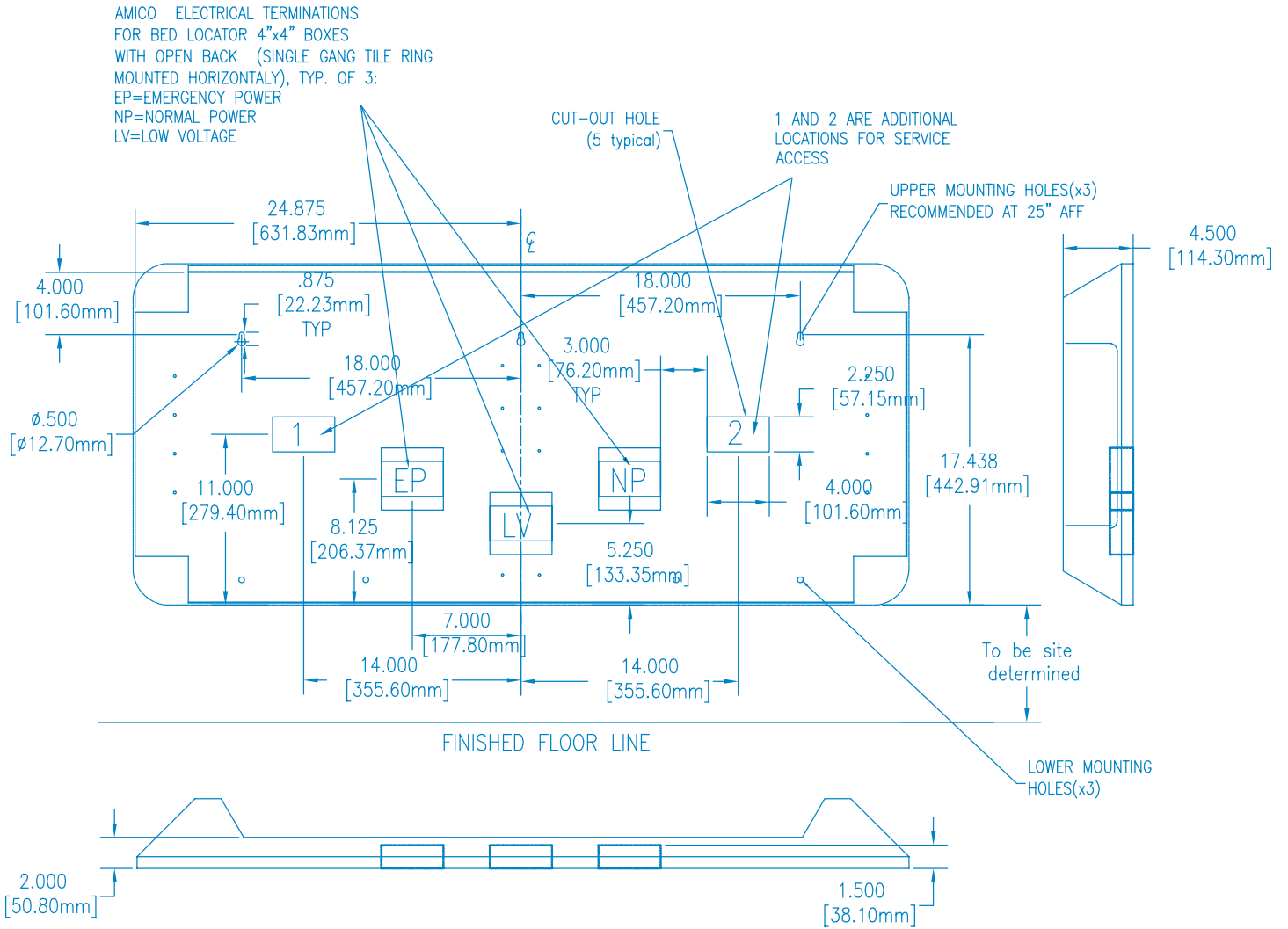



85 Fulton Way, Richmond Hill
Ontario, L4B 2N4, Canada

14 Madison Road, Fairfield
NJ 07004, USA

Toll Free Tel: 1.877.462.6426
Toll Free Fax: 1.866.440.4986
Tel: 905.764.0800 | Fax: 905.764.0862
www.amico.com

