



# Installation Instructions

## reflection series

LDR (labor/delivery/recovery) headwall




**caution**

Do not rest the cabinet on its movable bottom tray.  
Always lay cabinet on its back to avoid damage.

- 1 Secure the LDR metal frame to the wall structure (see [Figure 1](#)). Use hardware (by others) compliant to applicable building codes. Holes in mounting frame are to be drilled according to wall structure (stud) location.


Figure 1

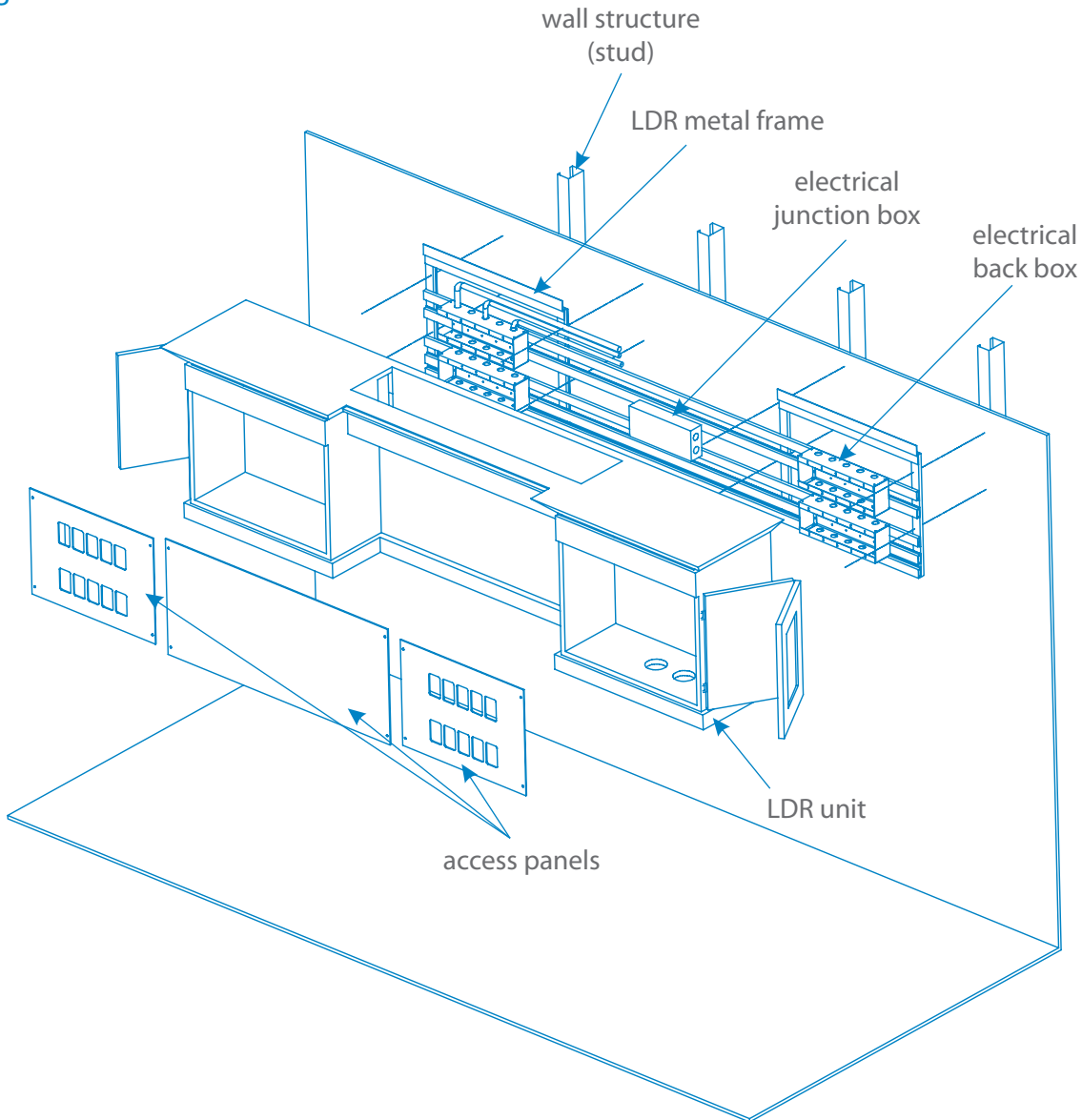

- 2 Make all rough-in connections between wall and LDR frame for piping and electrical services. Electrical services are terminated at junction boxes, and gas terminations are located according to rough-in drawings (provided) to facilitate installation.

- 3 Remove center electrical access panel (see Figure 2).

Figure 2

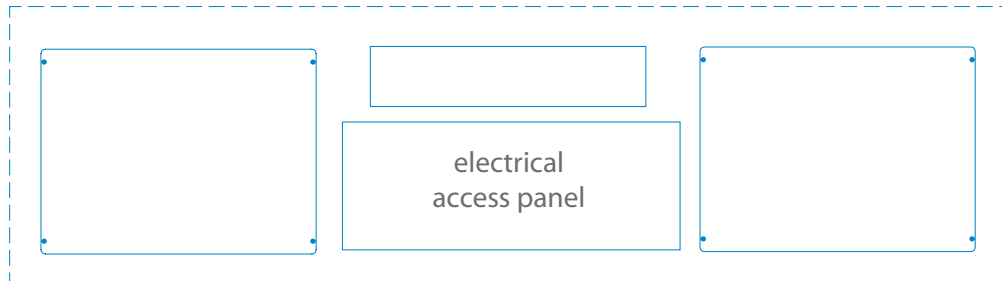

- 4 Hang the cabinet over the frame. Center the cabinet on the frame. Cabinet openings on the left and right sides should match with boxes on the frame.
- 5 Make all connections between light (if applicable) and bottom cabinet tray micro switches. Wiring is labeled to facilitate electrical connections.
- 6 Have left and right front access panels for your unit ready. Pull all electrical devices out of the boxes. Put one access panel on the cabinet and match the cut-outs with the device positions. Repeat for the other panel. Adjust cabinet position, if needed, to ensure proper alignment.
- 7 Holding the front panel in the correct position, surface mount the devices in place and then secure the cover to the cabinet. Drill pilot holes and attach with hardware provided.
- 8 Install device cover plates.
- 9 Anchor the cabinet to the frame using the middle brackets attached to the frame.
- 10 Install center electrical access panel.
- 11 Insert trim caps where necessary.
- 12 Install any accessories using hardware provided (if applicable).



85 Fulton Way, Richmond Hill  
Ontario, L4B 2N4, Canada

14 Madison Road, Fairfield  
NJ 07004, USA

Toll Free Tel: 1.877.462.6426  
Toll Free Fax: 1.866.440.4986  
Tel: 905.764.0800 | Fax: 905.764.0862  
[www.amico.com](http://www.amico.com)

