



# Installation Instructions

**regal series** recessed flatwall  
with studs



## Before Installation

### note

Flatwall installation should conform to local building codes

1 Review a copy of the final approved shop drawing(s) and/or submittal package. These documents will provide you with technical information specific to your installation, such as:

- equipment types & quantities
- room numbers and location
- shop drawings of each unit type
- wiring diagrams and rough-in locations

2 Locate the flatwall unit from the shipping boxes that you wish to install. The contents of each shipping box are listed on the shipping document(s).

- 1 Remove laminate panels from flatwall (using tool provided) by lifting panels vertically until resistance is felt, then rotate bottom of panel and pull down. Be careful not to drop panel.




tool provided:  
suction cup



**note**

Framing not shown

- 2 Secure flatwall and framing to building structure using hardware compliant to applicable building codes (by others). Ensure mounting location corresponds to on-site location of rough-ins.

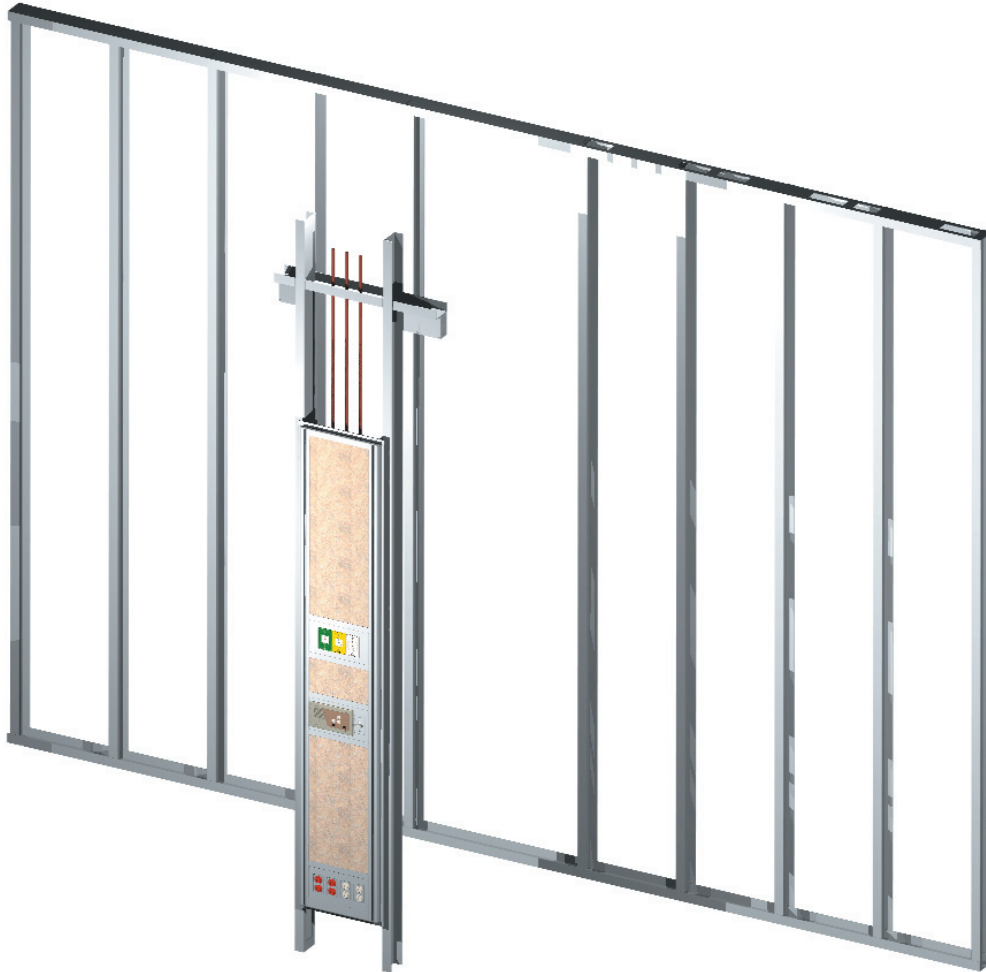


- 3 Make all electrical and piping connections. Med gas piping is terminated as per shop drawings; electrical services are terminated at boxes attached to upper framing in flatwall structure.

- 4 Install drywall. Note that the unit is shipped with a gap allowed between the flatwall front lip and stud face. This is to allow clearance for drywall installation.

- 5 Once drywall is installed, loosen flatwall frame to stud screws (see figure) and move flatwall frame so that the lip rests against the drywall. This will produce a finished appearance.

Figure A

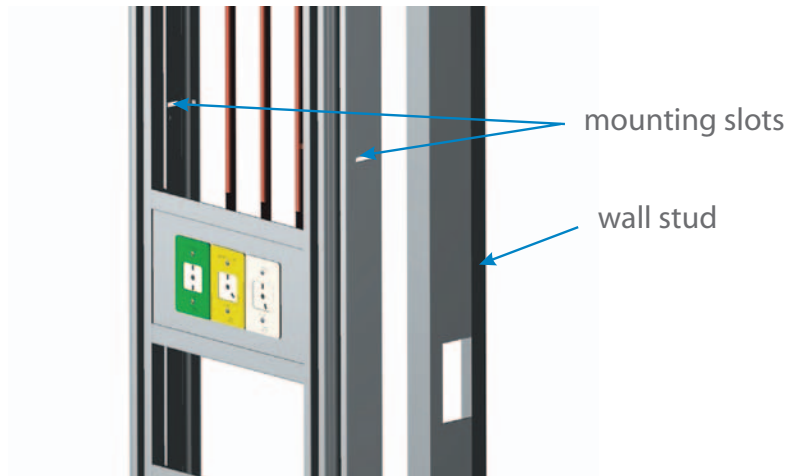

- 6 Replace laminate panels removed in [step 1](#).

## Other Installation Notes

- 1 Division 15 to connect gas stubs on flatwall to building medical gas supply lines previously roughed in.
- 2 Division 16 to feed flexible electrical conduits through top of flatwall and into ceiling space for connection to normal, emergency and low voltage boxes provided in ceiling space by division 16 contractor.



85 Fulton Way, Richmond Hill  
Ontario, L4B 2N4, Canada

14 Madison Road, Fairfield  
NJ 07004, USA

Toll Free Tel: 1.877.462.6426  
Toll Free Fax: 1.866.440.4986  
Tel: 905.764.0800 | Fax: 905.764.0862  
[www.amico.com](http://www.amico.com)

