

## Ohmeda Gas Control Panel Retro-Fit Manual



Ohmeda Gas Control Panel.  
Remove screws from panel.



Remove all internal components.



After removing all internal parts the only part left in the box should be the 5/16" male fittings for the input and output.



Mount the filler frame to the wall using the appropriate hardware for the type of wall you are mounting to. There are 6 holes provided for mounting the frame, 3 on top and 3 on the bottom.

Note: The frame can not be mounted on the wall centered on the existing back box. The retrofit is shipped with the filler frame attached to the control panel. Before you take these apart, push the assembly into the existing back box and position it into place and mark off the corners of the frame on the wall. These lines will be your reference for mounting the frame to the wall.



This is the input pigtail; it is already secured to the Amico control panel. Secure these fittings to the female end of the pigtail.



The pigtail that is supplied separately with the retrofit kit is for the output. Secure these fittings to the male end of the pigtail. The female end should be secured to the output on the back of the Amico control panel.



Secure the input and output pigtails to the existing connections in the back of the Ohmeda box. Please note on the next picture the position of the pigtails.



This should be the position of the pigtails as the Amico control panel is positioned into place and secured to the filler frame. It is very important that the pigtails be situated this way to give them enough room to flex, otherwise it is possible that the pigtail might kink.





85 Fulton Way, Richmond Hill, ON, L4B 2N4, Canada  
14 Madison Rd., Fairfield, NJ, 07004, USA

Phone: 1-877-GO AMICO (1-877-462-6426)  
Fax: (905) 764-0862 [www.amico.com](http://www.amico.com)