





**PURITAN-BENNETT
CONTROL PANEL
RETRO-FIT MANUAL**

Before

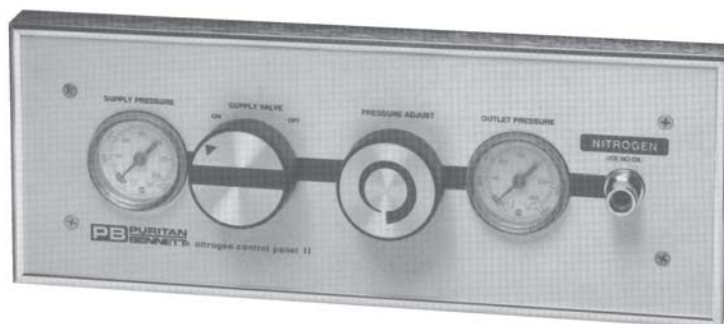


After

Puritan-Bennett Control Panel.
Remove screws from panel.

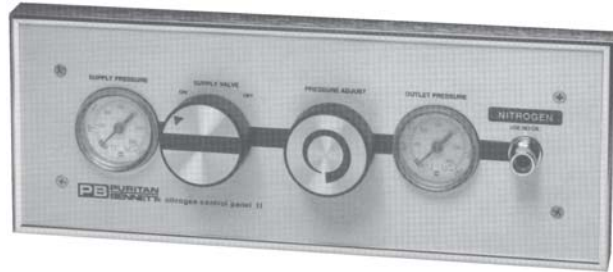

Remove all internal components.

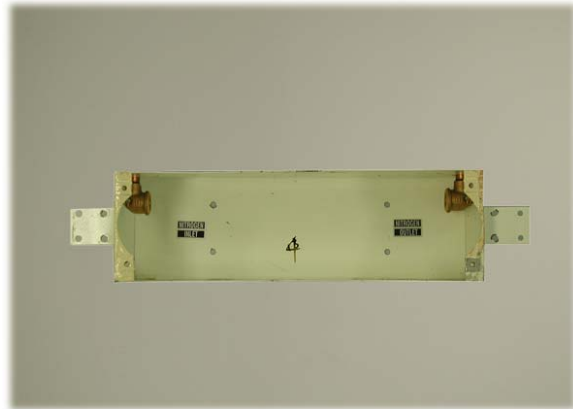


Mount the filler frame to the wall using the appropriate hardware for the type of wall you are mounting to. There are 6 holes provided for mounting the frame.

Note: The frame can not be mounted on the wall centered on the existing back box. The retrofit is shipped with the filler frame attached to the control panel. Before you take these apart, push the assembly into the existing back box and position it into place and mark off the corners of the frame on the wall. These lines will be your reference for mounting the frame to the wall.



Install the male elbows into the threaded connections in the back box. Connect the input tube into the appropriate fitting in the back box as shown.

Push the whole assembly gently into the back box and secure the panel to the filler frame using the #8 stainless steel machine screws provided with the kit. Please note the position of the pigtails.

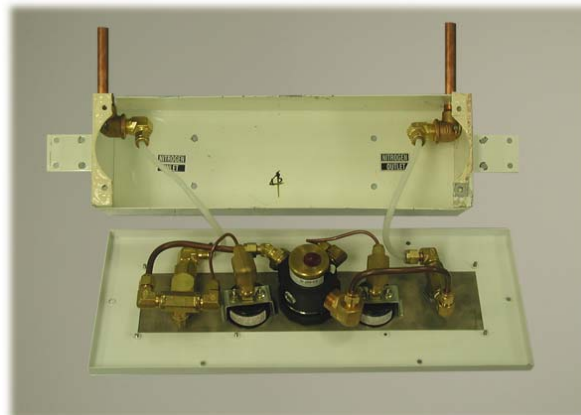



14 Madison Rd., Fairfield, NJ 07004 USA
85 Fulton Way, Richmond Hill, ON L4B 2N4 Canada

Tel: 1-877-GOAMICO (1-877-462-6426)
Fax: (905) 764-0862 www.amico.com